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Operating Instructions

Mode d'emploi

Istruzioni d'uso

Manual de instrucciones

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Bruksanvisning

Bruksanvisning

Käyttöohje

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DE

EN

FR

IT

ES

NL

DK

SE

NO

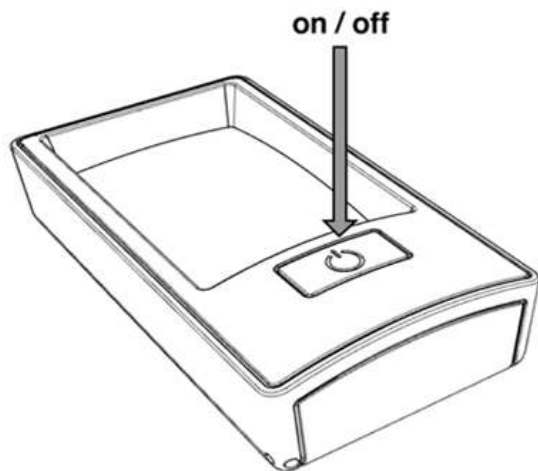
FI

PL


CZ

JP

FRONT

BACK

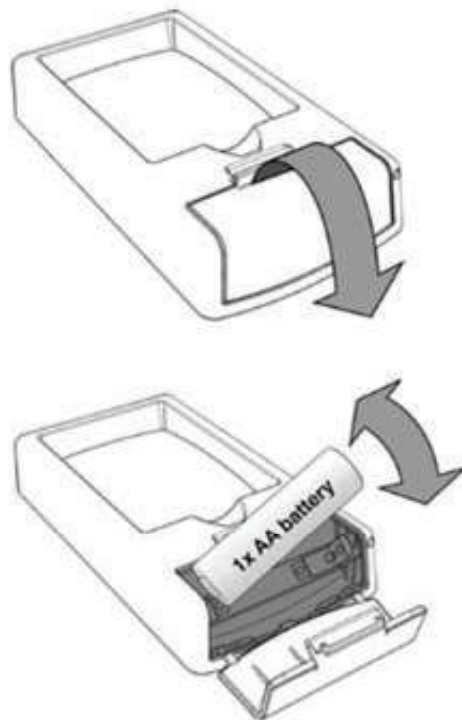Deutsch

DE

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English

About This Operating Instructions

EN

Thank you for choosing an ocularX magnifying glass. Please read the operating instructions and safety notes carefully before first use. These instructions contain important information regarding the safe use, cleaning, and disposal of the product. Keep these operating instructions for future reference. If other persons use this magnifying glass, please make these instructions available to them.

The Operating Instructions are applicable for the following products:

Item No.	Description	Magnification	Dioptre	Lens size
XM0101	M1 LED 2-4x	2,0x & 4,0x	3,0 & 15,0 D	114x70mm
XM0102	M1 LED 2,2x	2,2x	5,0 D	76x50mm
XM0103	M1 LED 2,5x	2,5x	10,0 D	Ø58mm
XM0104	M1 LED 3,5x	3,5x	13,5 D	Ø58mm
XM0105	M1 LED 4,5x	4,5x	17,5 D	Ø58mm

Intended use

The hand-held magnifying glass is used for the **optical magnification** of objects or texts in **close range**, as well as for craft work, DIY or repairs. Integrated lighting (LED) aids in illuminating the viewing area.

Safety instructions

Important information for your safety

Read and follow the following safety instructions carefully!
Make other people aware of these dangers.

- **Risk of glare and injury:** Never look directly at the sun or at very bright light sources through optical lenses.
- Do not shine directly into an eye, and never look into the LED lighting from a short distance.
- **Fire hazard due to magnifying glass effect:** Do not point the magnifying glass at the sun or very strong light sources. Do not leave the magnifying glass in direct sunlight without a cover.
- Use the magnifying glass only in **dry** rooms.
Do not immerse in water and **do not** clean under running water.

- Protect the product against **shocks, impacts and moisture** and **excessive heat**. Do not leave the magnifying glass on radiators or in direct sunlight.
- This is not a toy: Children may use the product only under supervision.
- Avoid opening, repairing, or modifying the magnifier.

Commissioning

Inserting/replacing batteries (see diagram on page 2)

- Open the battery compartment cover.
- Remove any spent batteries, if applicable.
- Insert new batteries/rechargeable batteries, ensuring correct polarity.
- Close the battery compartment cover.

Batteries – Safe handling

- When inserting the batteries, ensure **correct polarity** (Plus/Minus). Incorrect polarity or improper insertion can damage electronic components.
- Only use AA batteries (1,5 volts) or compatible rechargeable batteries (1,2 volts).
- Do **not use** damaged, deformed, or leaking batteries.
- Do not short-circuit, crush, disassemble, open, heat, or throw batteries into a fire.

- Do **not** charge non-rechargeable batteries.
- Keep batteries out of the reach of children (risk of swallowing).
- In case of contact with battery fluid: Immediately rinse skin/eyes with plenty of water and seek medical advice if irritation occurs.
- Remove the batteries if the product will not be used for an extended period.

Using the magnifying glass

The magnifying glass can be used with or without illumination. To switch on the illumination, press the button. To switch it off, press the button again.

Troubleshooting

- **Illumination not working:** Battery missing/empty → Insert or replace battery; check polarity.
- **Illumination is dim/flickering:** Battery nearly empty or contact loose → Replace battery; check battery contacts and clean carefully.
- **Image is blurred:** Lens dirty → Clean the lens with a soft, lint-free cloth (e.g., a glass cleaning cloth).
- **Moisture in the device:** Turn off the lighting, remove the battery, and allow it to dry completely at room temperature; only then should you use it again.

Care instructions

Switch off the lighting before cleaning. Clean the lens and housing with a soft, lint-free cloth. In case of heavier soiling, slightly dampen the cloth.

Do not use / avoid: Abrasive cleaning agents, alcoholic or organic solvents, cleaning under running water, immersion in water. Failure to observe these instructions may damage the product.

Disposal

This product is subject to the Waste Electrical and Electronic Equipment (WEEE) Directive, in its currently valid version.

Do not dispose of the product with household waste; instead, dispose of it via approved collection or disposal centres in accordance with the regulations applicable in your country.

Disposal of batteries and rechargeable batteries

Batteries and rechargeable batteries must not be disposed of with household waste. Remove the batteries/rechargeable batteries and deposit them at an official collection point.

Conformity

The product meets all requirements of the applicable UK directives and national regulations. We confirm this through the UKCA marking. You can access the current Declaration of Conformity at the following website:

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If an older model is no longer listed here, please contact info@eschenbach-optik.com.

Warranty

Within the scope of statutory provisions, Eschenbach Optik GmbH guarantees the functionality of the product described in these instructions against defects attributable to manufacturing or material errors.

No warranty claims can be accepted for damage that arises due to improper handling, including damage from falls or impacts. Warranty claims may only be made upon presentation of the proof of purchase.

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